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Configuration Inspire X

Activer Alerte auto

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The screenshot displays the 'Auto Fall Alert' configuration screen in the Inspire X software. The interface is divided into several sections:

- Navigation Menu (Left):** Includes 'Latest News', 'Pre-Fitting', 'Get Started', 'Fitting' (with sub-items like QuickFit, Fine Tuning, User Controls, Experience Manager, Feedback Celler), 'Patient Tools' (with sub-items like Frequency Lowering, Sound Manager, Tinnitus, Memories, Accessories, Indicators, Fitting Summary, Data Log, Ready, Set, Hear, How Did We Do?), and 'Advanced Tools'.
- Top Bar:** Features 'Right Binaural Left' controls.
- Main Content Area:**
 - Self Check:** A plus sign icon.
 - Remote Programming:** A plus sign icon.
 - Auto Fall Alert:** A minus sign icon.
 - Auto Fall Alert:** A section with a red dot, a blue dot, and two checked checkboxes.
 - Fall Detection Sensitivity:** A section with a red dot, a blue dot, and three buttons: 'Low', 'Default', and 'High'. An information icon is next to 'Default'.
 - Fall Risk Screening:** A section with a red dot, a blue dot, and three questions with 'Yes' and 'No' checkboxes:
 - 'Have you fallen in the past year?' (Yes: , No:)
 - 'Do you feel unsteady when standing or walking?' (Yes: , No:)
 - 'Do you worry about falling?' (Yes: , No:)

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Device Guide

Customized for: Cassie Falls Session 5/2/2021

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➔ Your device features Automatic Alert. For additional information, refer to Fall Alert in the Thrive Hearing Control app or discuss with your hearing care professional.

Controls

Accessory Stream

To start or stop the accessory stream, firmly double tap directly on the top portion of either ear with either an open palm or two fingers. Your ear should push against your hearing aid with each double tap.

Memory

Press and release the upper or lower portion of the switch on either device to change the memory.

Memory	Environment	Indicator	Thrive/iOS
1	*Normal	Normal	Normal
2	Restaurant	Restaurant	Restaurant
3	Car	Car	Car
4	Disabled	Disabled	Disabled
🔊	Stream Boost: On	Tone Series	Stream Boost
Custom	Custom memory	Tone Series	Custom

Manual Alert

Press and hold the upper or lower portion of the switch on either device to trigger a manual alert.

Event	Indicator
Alert Sent	Alert Sent
Alert Received	Alert Received
Alert Cancelled	Alert Cancelled
Communication Failure	Tone Series

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